AccuFilm®

ARTICULATING PAPER & FILM

parkell®

A00013revJ0519

EN

PRODUCT DESCRIPTION:

Super-thin, high accuracy, moisture-resistant dental articulating products.

INDICATIONS:

For precise locating of occlusal prematurities in dental arches and tight spots on prosthetic contacts or inside crowns.

CONTRAINDICATIONS: None.**PRECAUTIONS:** None.**ADVERSE EVENTS:** None.**PROCEDURE FOR USE:**

- Dry dental or prosthetic surfaces before use.
- Use your regular articulating or crown seating technique. Use articulating forceps, if desired.
- For single-sided AccuFilm, the dull side is the marking side.
- For double-sided AccuFilm, both arches are marked simultaneously.

HINTS

- When equilibrating hard-to-mark surfaces (polished gold, etc.), applying a thin coat of petroleum jelly to the AccuFilm will often allow readable marks.
- For sheets, a lengthwise fold down the center will prevent curling.

AVAILABLE PRODUCTS:**■ SINGLE-SIDED ACCUFILM® CANISTER**

Roll of 280 pre-cut strips.

- Single-sided Black strips (REF S014)
- Single-sided Red strips (REF S015)
- Single-sided Green strips (REF S016)

■ DOUBLE-SIDED ACCUFILM® CANISTER

280 pre-cut strips.

- Double-sided Red/Black strips (REF S017)
- Double-sided Black/Black strips (REF S018)
- Double-sided Red/Red strips (REF S021)

■ DOUBLE-SIDED ACCUFILM® BOOKLETS

Box of 5 booklets with 50 sheets per booklet.

- Double-sided Red/Black strips (REF S053)

■ DOUBLE-SIDED ACCUFILM® ARCH BOOKLETS

Box of 6 booklets with 12 sheets per booklet.

- Double-sided Blue/Red Arches (REF S050)

■ DOUBLE-SIDED ACCUFILM® FASTCHECK

Box of 50 devices.

- Blue/Blue Strips (REF S054)

■ ACCUFILM® IV BRUSH-ON LIQUID

Fast drying, blue liquid (10 ml) with solvent (20 ml).

- 2 Bottles (REF Stock No. S020)

WARRANTY AND TERMS OF USE:For full Warranty and Terms of Use information, please see www.parkell.com.Safety Data Sheets (SDS) are available at www.parkell.com.

Parkell's Quality System is certified to ISO 13485.

EXPLANATION OF SYMBOLS USED

Follow instructions for use

Store product within min./max temperature range noted on packaging if applicable.

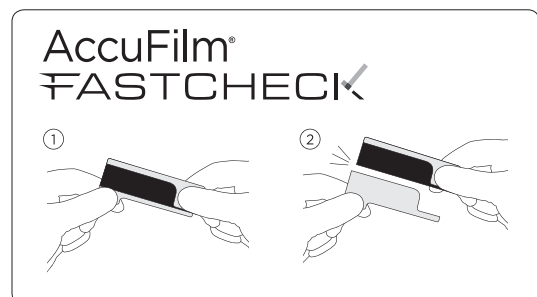
Store product tightly sealed away from direct sunlight

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Directa AB - P.O. Box 723, Finvids väg 8, 194 27 Upplands Väsby