parkell[®]

Absolute
DENTIN[®]

Dual-Cure Core Composite

Instructions for Use

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UKRP: Topdental (Products) Ltd,
12 Ryefield Way, Silsden, West Yorkshire BD20, OEF England UK

mt-g medical translation GmbH & Co KG
Stuttgarter Straße 155 • D-89075 Ulm

CE **USA**
1639 MADE IN USA A00361revN0821

EN

Rx Only—product is intended for use by a licensed dental professional. For Safety Data Sheet (SDS) go to www.parkell.com.

Description

Parkell’s Absolute Dentin® is a strong, dual-cure, composite resin material that is highly filled with fluoridated barium/aluminum glass for radiopacity. It is intended for core fabrication, and is supplied in two convenient packages: a 50 ml/110 gm cartridge and a 10 ml/22 gm cartridge, each to be expressed with the appropriate automix gun. The uncured material flows well and stacks without slumping, while the cured material “feels” much like dentin when prepared with rotary instruments.

Absolute Dentin works with all dentin/enamel bonding agents that are compatible with dual-cure or self-cure composites, including all of Parkell’s 4-META-based bonding products (Brush&Bond®, Brush&Bond® Universal, Amalgambond™ and C&B-Metabond®).

Absolute Dentin kits are available in three shades: Tooth Shade (similar to A2/A3), Arctic White (A1/B1), and Blue (easily distinguished from surrounding tooth structure). Each kit contains an appropriate number of mixing tips and intraoral tips.

Indications for Use

Absolute Dentin is a dual-cure, composite resin core build-up material that is usually used to restore missing tooth structure for which a dental restoration, usually a crown, is fabricated.

Directions

1. Apply the appropriate dentin/enamel bonding agent to the prepared tooth and cure it following the material’s instructions. Since Absolute Dentin is a dual-cure material, make sure that the bonding agent chosen is compatible with DUAL-CURE or SELF-CURE composites. Certain non-Parkell self-etching bonding agents may interfere with the setting ability of dual-cure composites, so check the instructions for your bonding agent before proceeding.

2. Setting up the product for use:

a. If using the 50 ml automix cartridge, insert it into any DS-50 (1:1 / 2:1) impression gun such as Parkell’s ☞ S343. Remove and discard the blue sealing cap or used mixing tip from the cartridge, and express out a small amount of material through both round orifices. Then attach a yellow mixing tip to the cartridge, and a black intraoral tip to the mixing tip.

b. If using the 10 ml automix cartridge, insert it into the appropriate 10 ml (1:1) impression gun such as Parkell’s ☞ S327. Remove and discard the brown sealing cap or used mixing tip from the cartridge, and express out a small amount of material through both round orifices. Then attach a brown mixing tip to the cartridge, and a clear, angled intraoral tip to the mixing tip.

3. Express out a pea-sized amount of Absolute Dentin through the tip onto a pad to ensure a complete mix of base and catalyst pastes. Immediately express the mixed material directly from the cartridge onto the bonding agent coated tooth. A matrix band or core-forming capsule may be used to contain the material.

4. Absolute Dentin will self-cure without light in about 4 minutes. The light initiator in the product is camphorquinone (CQ), which has an activation range of between 460 and 480nm. Absolute Dentin will light-cure to a depth of 3mm with a 40 second exposure to any dental curing light producing an intensity of at least 400 mW/cm². Actual clinical depth of cure will depend on factors such as time, light intensity, device condition, distance between the light and the restoration, and material shade.

5. Once fully cured, prepare the core as usual with rotary instruments of choice.

6. Leave the used mixing tip in place on the cartridge to serve as the sealing cap between uses. The cartridge, used mixing tip and gun may be disinfected with appropriate dental surface disinfectants. Check with the disinfectant manufacturer before use.

Storage and Use

Store the material in a cool, dry place. For longest shelf life, refrigeration without freezing is preferred. After removal from the fridge, allow Absolute Dentin to warm to room temperature for best handling.

Cold material may be difficult to express and take longer to cure.

Contraindications

Absolute Dentin should not be used on or by persons who are methacrylate sensitive. Uncured material should not be swallowed.

Tips for Dental Practitioners for Cross-contamination Control

Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient; use new, uncontaminated gloves when handling multiple-use dental dispensers; utilize dental assistants to dispense material for the dentist; avoid contact of the reusable parts (e.g., the body of the multiple-use dental dispenser) with the patient’s mouth; do not reuse the multiple-use dental dispenser if it becomes contaminated; do not reprocess a contaminated multiple-use dental dispenser by using chemical wipes or disinfectants; do not immerse multiple-use dental dispensers in a high level chemical disinfectant, as this may damage the dispenser and the material contained in the device; do not sterilize multiple-use dental dispensers, as this may damage the material contained in the device.

Warranty and Terms of Use

For full Warranty and Terms of Use information, please go to www.parkell.com. Safety Data Sheets (SDS) are available at www.parkell.com. Parkell’s Quality System is certified to ISO 13485.

Available Kits and Refills

Absolute Dentin 50 ml Kits

Include (1) 50 ml split cartridge (110 gm / paste total) of one shade, (40) mixing tips and (40) intraoral tips:

- ☞ S300 – Arctic White Shade
- ☞ S301 – Tooth Shade
- ☞ S307 – Blue Shade

Absolute Dentin 50 ml Accessories:

- ☞ S302 – Yellow 50 ml Mixing Tip Refill
- ☞ S308 – Dark Grey Intraoral Tip Refill (use with ☞ S302)
- ☞ S343 – 1:1 / 2:1 50 ml Split-Cartridge Dispensing Gun

Absolute Dentin 10 ml Kits

Include (2) 10 ml cartridges (20 ml - 44 gm / paste total) of one shade, (16) mixing tips and (16) intraoral tips:

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