

# DisCide<sup>®</sup> ULTRA Surface Disinfectant Instructions

If gross debris is present on the surface


CLEANING before disinfecting\*



**Step 1: Spray** hard non-porous surface with DisCide Ultra Spray to help loosen bioburden.



**Step 2: Wipe** down the surface with a DisCide Ultra towelette to remove debris and bioburden.

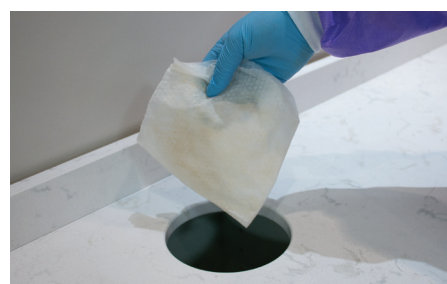

**Step 3: Dispose** of used DisCide Ultra towelette.

\* If less than 5% bioburden present, 1-step cleaning and disinfecting can occur.

## DisCide ULTRA Product Line

### TOWELETTES



Ref# **10DIS**  
10" x 10" towelettes  
60 count canister



Ref# **60DIS**  
6" x 6 3/4" towelettes  
160 count canister

### SPRAYS



Ref# **3565Q**  
Quart Sprayer



Ref# **3564**  
4 oz. Bottle

### REFILLS



Ref# **3565G**  
Gallon Refill

### ACCESSORIES



Ref# **3567**  
Empty quart bottle  
with sprayer and  
DisCide ULTRA label




Ref# **3567T**  
Quart spray trigger



Ref# **3519**  
Unlabeled empty  
quart bottle with  
sprayer

If NO gross debris is present on the surface

DISINFECTING after cleaning\*

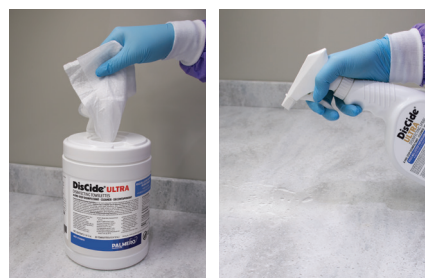


**Step 1: Use** DisCide Ultra towelette or **Spray** DisCide Ultra Spray on the pre-cleaned surface.



**Step 2: Leave** the surface wet for one minute.



**Step 3: Wipe** down the surface with a DisCide Ultra towelette to remove any residue chemical left behind.

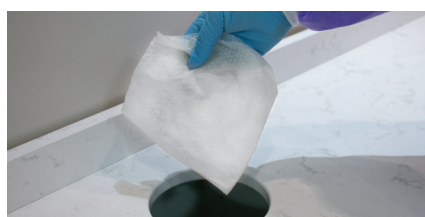

**Step 4: Dispose** of used DisCide Ultra towelette.



For more information, visit [palmerohealth.com](http://palmerohealth.com), email [customerservice@palmerohealth.com](mailto:customerservice@palmerohealth.com) or call 800.344.6424.

Palmero is now a proud member of HuFriedyGroup

